


### Avsedd användning

Polstring är avsedd att använda med tillbehör Fotstödssida från HD Rehab.

### Ingående delar

2 x Polstring  
(höger och  
vänster)

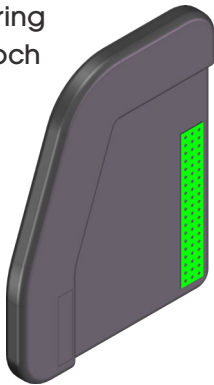


2 x Kardborrband  
mjuk sida 100 mm



2 x Kardborrband  
mjuk sida 60 mm



### Montering

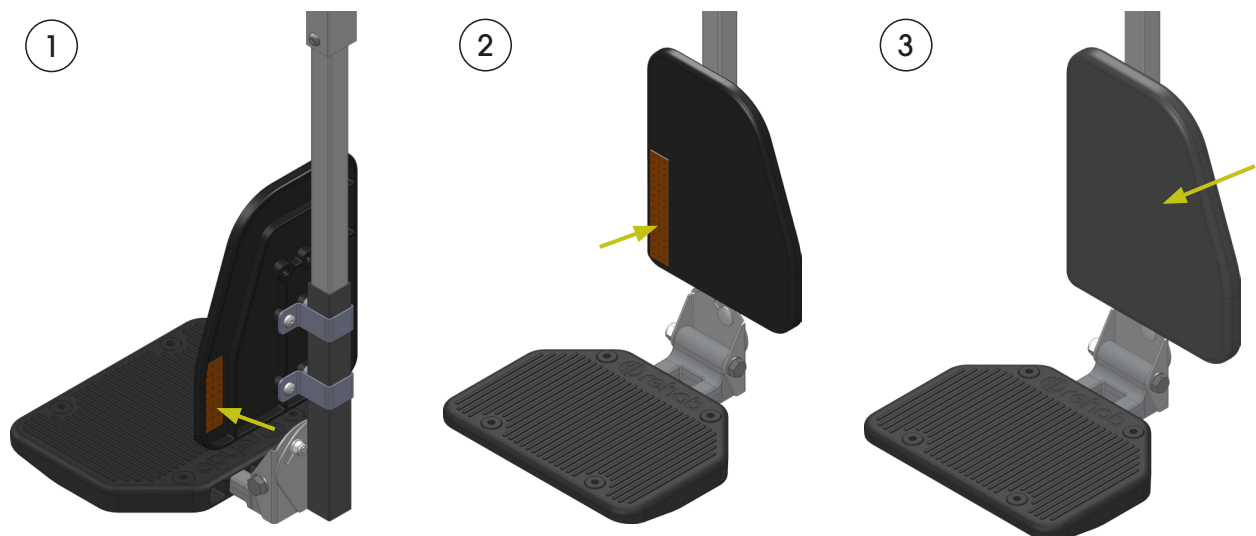

- ① Fäst den kortare kardborrband på fotstödssidans baksida.
- ② Fäst den längre kardborrband på fotstödssidans framsida.
- ③ Passa polstring över fotstödssida och fästa med kardborrband på båda sidor.
- ④ Upprepa dessa steg på andra sidan.

### Användning och skötselråd


Polstring till fotstödssida kan rengöras genom avtorkning med tvål och vatten. Byt alltid ut skadade delar omedelbart. För övrig information se *Underhållsanvisning*, dokument nummer 95730.

### Intended use

Padding is intended to be used with the accessory Footrest side support from HD Rehab.

### Parts included

2 x Padding  
(LH and RH)

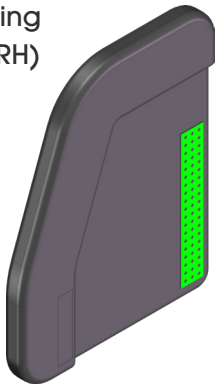


2 x velcro, soft  
side 100 mm



2 x velcro, soft  
side 60 mm



### Assembly

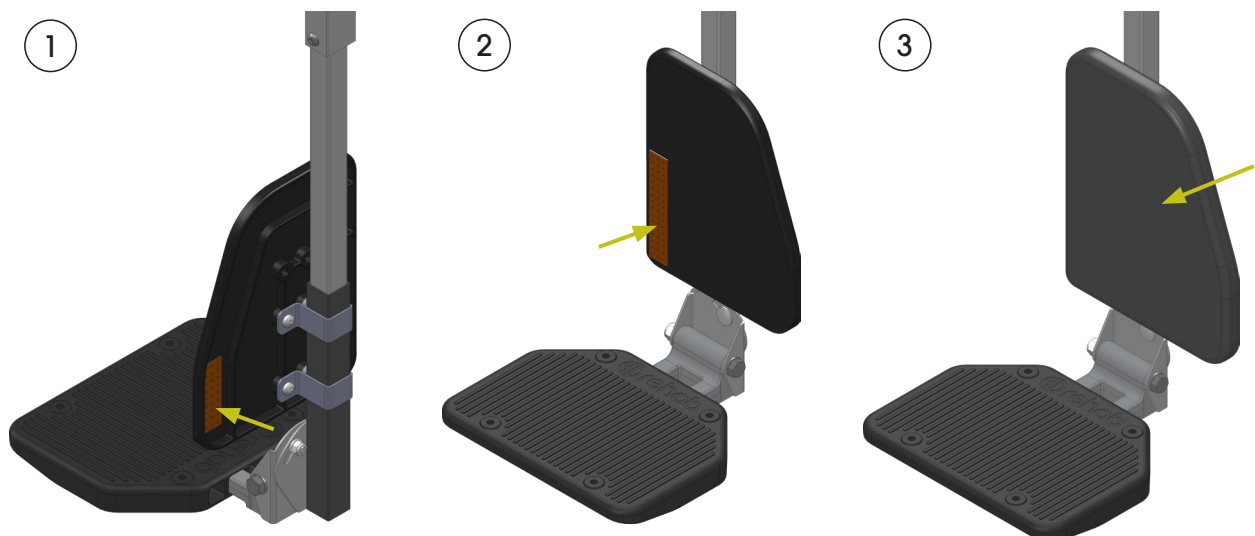

- ① Attach the shorter velcro piece to the back side of the footrest side.
- ② Attach the longer velcro piece to the front side of the footrest side.
- ③ Fit the padding over the footrest side and secure it with the velcro on both sides.
- ④ Repeat these steps for the other footrest side.

### Care instructions

The footrest side padding can be cleaned by wiping with soap and water. See *Maintenance Instructions*, document number 95730-1. Always replace damaged parts immediately.